



MEDintim
personal healthcare



Erectionrings

For longer and stronger erections.

Instruction to use:

At first pull your testicles, followed by the non erected penis through one of the rings. Pull the ring toward the body. Choose your personal right size. The right size should be tight, but not hurt you. Once erected, the backflow of the blood at your penis is retarded - This effect increases your erection and gives both partners more sensation. Woman may be stimulated at the clitoris by the well positioned ring. Even after your climax, the penis could still be erected for a while keeping his shape.

Attention:

Please note: Too long reduced blood flow could hurt you and be painful. If you have any irritations, please do not use this device.

- 3 rings
- Made of natural latex rubber
- Color: blue
- Inner circle: 32, 38, 50 mm
in inches: 1.25, 1.5, 2 inch

Art. No.: MI ER
EAN 40 13273 000734
PZN 6681998



Kessel Marketing & Vertriebs GmbH
Kelsterbacher Straße 28
64546 Mörfelden-Walldorf, Germany

Tel. +49 (0) 6105 20 37 20
Fax +49 (0) 6105 45 59 01
E-Mail: service@medintim.de

www.medintim.de