



MEDintim
personal healthcare



Oralsafe

protects at oral-genital contacts

The new oral safe latex sheets are the best protection for oral-genital contact.

instruction to use:

open carefully the foiled latex tissue. Lubricate the vaginal or anal part with a waterbased lubricant. Place well the Oral Safe tissue to cover the vagina with clitoris or anus. Fix the tissue at its place while you have oral-genital contact. Dispose the used tissue at your waste. Don't flush into the toilette.

Ingrediens:

natural latex rubber

Contents:

1 latex tissue for single use,
available without flavour or with strawberry or
vanilla aroma
size: 252 x 115, colour: beige
PU: 10 units

Storage:

Keep dry and cool.

Manufacturer:

Kessel Marketing & Vertriebs GmbH

Strawberry EAN 4013273900164

Vanilla EAN 4013273900171

Neutral available on request



Medical Device Class I



Kessel Marketing & Vertriebs GmbH
Kelsterbacher Straße 28
64546 Mörfelden-Walldorf, Germany

Tel. +49 (0) 6105 20 37 20
Fax +49 (0) 6105 45 59 01
E-Mail: service@medintim.de